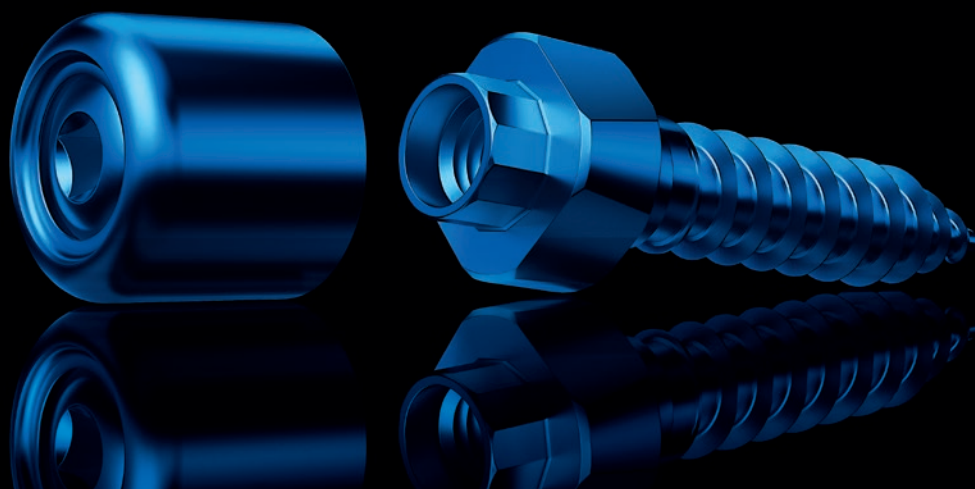
BENefit[®].KFO

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Skeletal anchorage in orthodontics

An important requirement for successful orthodontic treatment is stable anchorage. Conventional orthodontic anchorage can be limited by periodontal disease and/or missing teeth. Skeletal anchorage has been proven to be very effective. It can be an essential part of non-compliance therapy and broadens the options available to the orthodontic practitioner.

Mini-Implants are especially advantageous because of the minimally invasive insertion protocol and their relatively low cost.

However, limitations can occur when conventional Mini-Implant Systems are used for orthodontic anchorage. In collaboration with leading specialists, PSM has developed the unique **BENEFIT** TAD System for temporary fixation in the palate. **BENEFIT** offers high precision titanium implant screws and a large variety of prefabricated connecting parts such as **BENEplates**, abutments and palatal expanders.

With the introduction of the **BENEFIT** System PSM has considerably enhanced the therapeutic diversity of the treatments available for Mesialization, Distalization, RPE as well as uprighting or anchorage of the dentition.

With **BENEFIT**, PSM was the first company to offer a full range of skeletal anchorage appliances for the palate – a trend that has become state-of-the-art in Orthodontics.

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EINFÜHRUNG

■ Note:

All products marked with **QB** can be used with **quattro**® and **BENEFIT**® screws!

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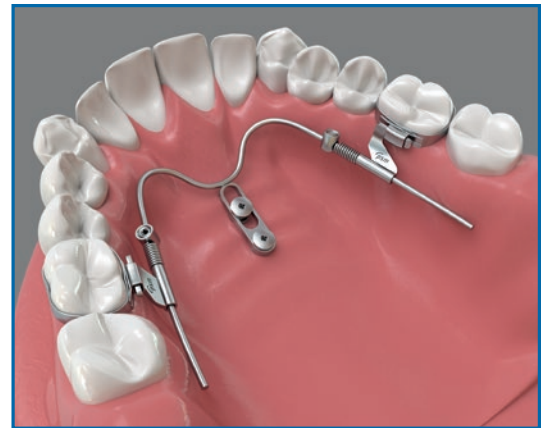
BENeslider Video


The BENeslider

The most classical indication for skeletal anchorage using Mini-Implants with abutments is molar-distalization and/ mesialization in the maxilla. By means of the so called "**BENeslider**" the molars can be bodily distalized and/or mesialized using the Mesialslider. In many cases, tooth extraction or dental implants can be avoided.


To couple two **BENefit®** Mini-Implants, a **BENEplate** with welded wire is connected to the Implants and the **BENeslider** parts (mobilizer, springs, **BENetube**) are placed on the wire. Active force is applied by springs (240 g or 500 g) and activated with the Mobilizer.

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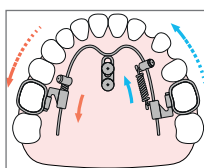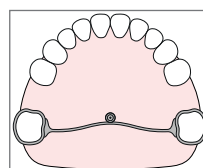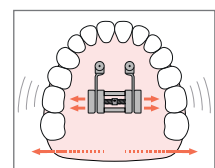
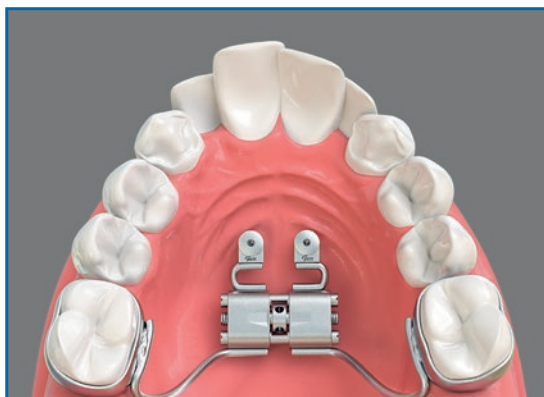
Hybrid Hyrax Video

The Hybrid Hyrax®

The new Hyrax® Ring and Hyrax® Abutment (page 9) facilitate the construction and fixation of a Hybrid Hyrax® appliance. It allows welding or soldering with the Hyrax® Screw as well as little rotation movements when connecting the appliance to the BENEFIT Screws. The Hyrax® Ring is supplied with 1 fixation screw each.

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The bmx – BENEFIT Maxillary Xpander

Additionally to the Hybrid Hyrax that is available for some years now the Orthodontists expressed their desire for an appliances that is exclusively anchored on mini-implants.

The new bmx System now offers the possibility for an enforced, non-surgical RPE with adolescent patients up to 13 years based on 2 paramedian anchored **BENEFIT** miniscrews.

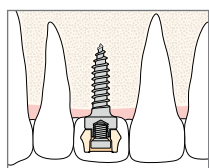
The pre-fabricated **bmX** expanders with 6 mm, 8 mm and 10 mm distance can be installed in the palate without an impression or additional laboratory work using the De Franco Parallel Guide (page 18).

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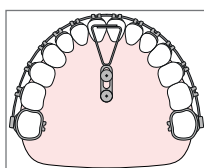
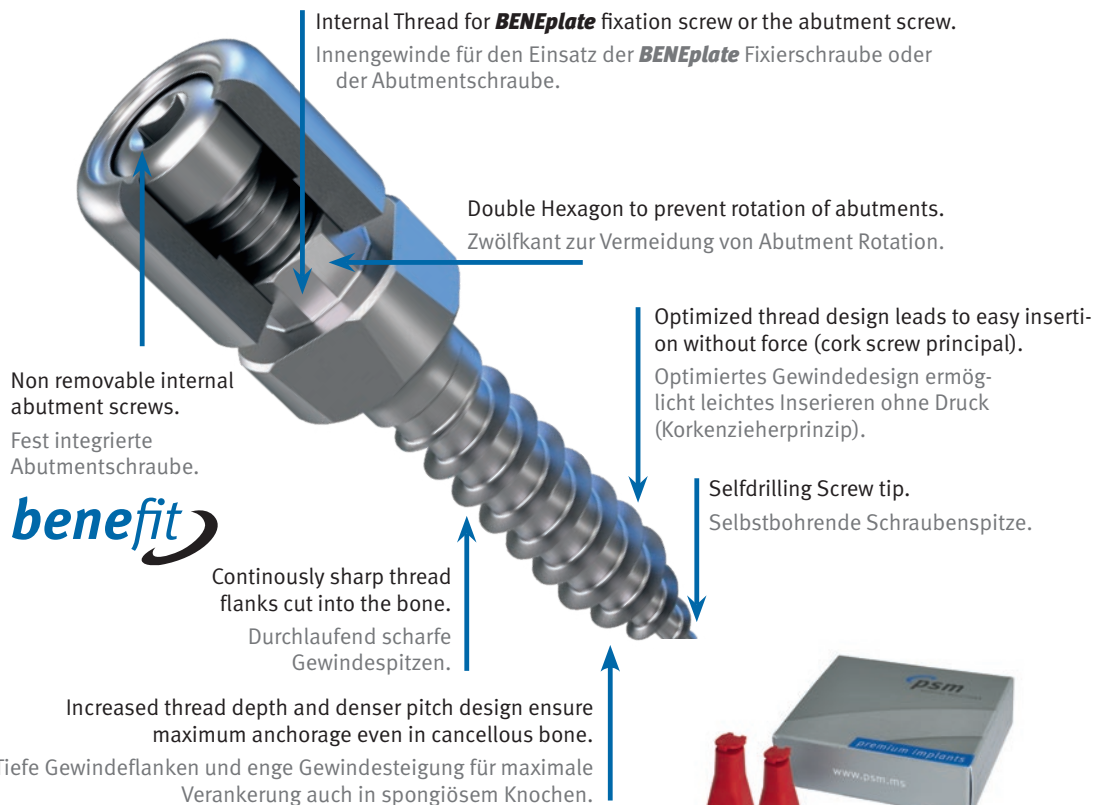
Packaging: 1 ea.

VPE: 1 Stk.

Direct screw removal from the sterile holder.

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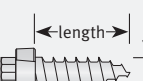
ST

Note:

All implants are supplied in sterile single use packaging.

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BENEFIT® Orthodontic Screw 2.3 mm, Material Titanium

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33-54512 20 mm
33-54515 30 mm
33-54518 50 mm

DENTAL
DENTAL
DENTAL

Screwdriver insert e.g. for abutment retaining screws, **BENEplate**
fixation screws and Mobilizer activation

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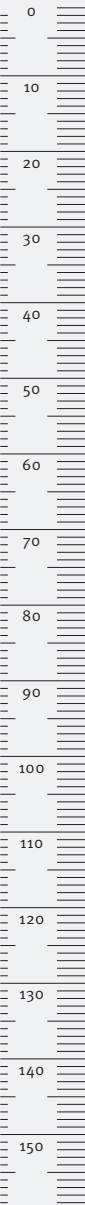
QB DENTAL

Extension, +16 mm
Verlängerung, + 16 mm

The extension of 16 mm works for all rotary instruments of the
quattro® and **BENEFIT®**-System with a latch type DENTAL attachment
(max. rpm 600).

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BENEFIT® Abutment with brackets

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33-54445

Abutment with loop acc. to Dr. S. Renger,
St. Louis, France

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Internal Thread for **BENEplate** fixation screw
or the abutment screw.

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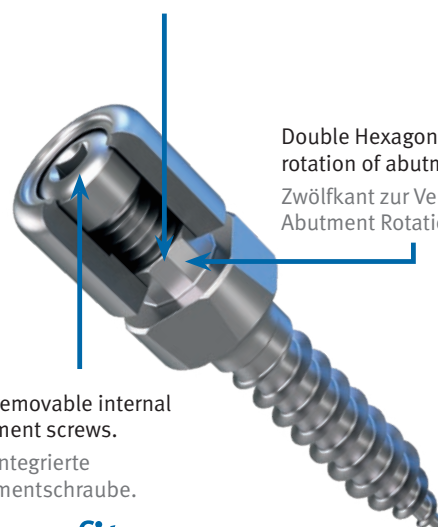
Double Hexagon to prevent
rotation of abutments.

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Non removable internal
abutment screws.

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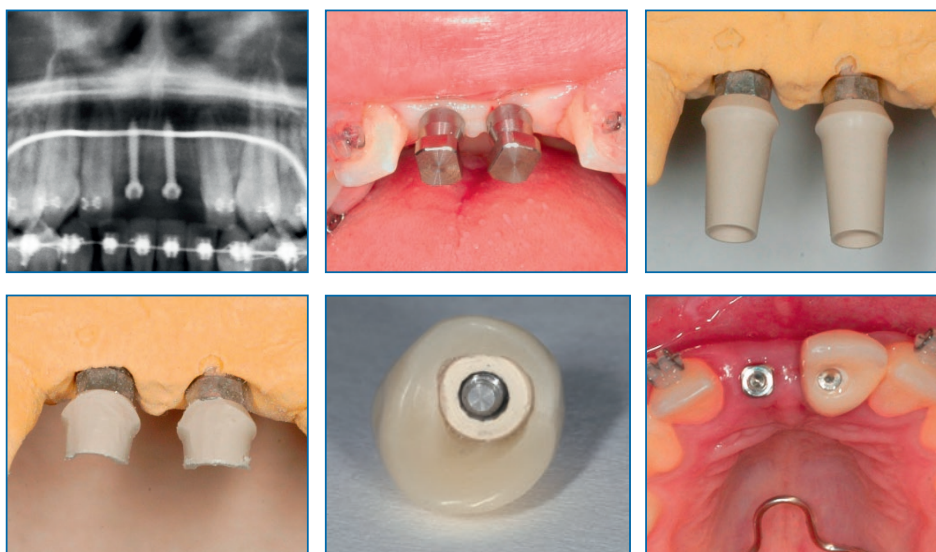
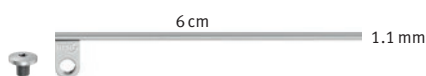
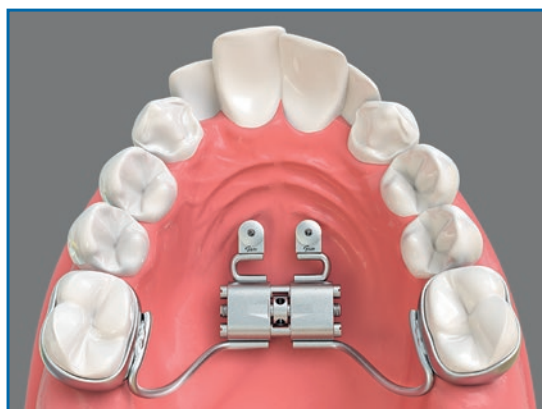
33-54463

BENEFIT® Hyrax® Abutment, 1 ea
BENEFIT® Hyrax® Abutment, 1 Stk.

The new Hyrax® Ring and Hyrax® Abutment facilitate the construction and fixation of a Hybrid Hyrax® appliance. It allows welding or soldering with the Hyrax® Screw as well as little rotation movements when connecting the appliance to the BENEFIT Screws. The Hyrax® Ring is supplied with 1 fixation screw each.

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HYBRID HYRAX RPE HYBRID HYRAX GNE

■ **Note:**
Further Hyrax Ring
versions see on page 15.

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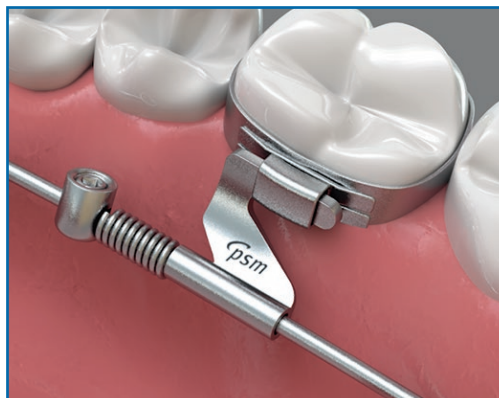
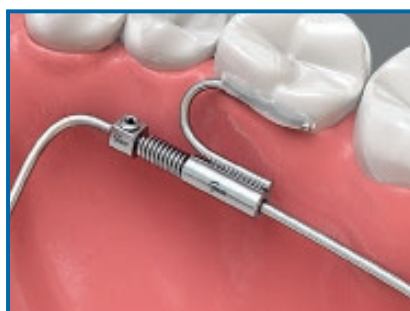
33-54545

Mesialtube acc. to Dr. Banach with wire 1.1 mm, 40 mm and hook

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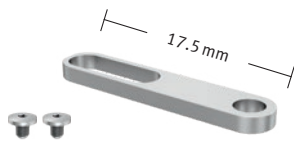
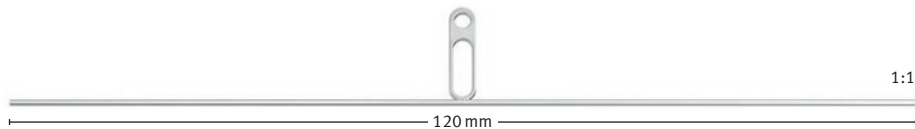
■ **Note:**

All **BENEplates** are supplied with 2 fixation screws

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BENeplate

33-54472 1.1 mm stainless steel

BENeplate, long, with 1.1 mm lateral wire (12 cm)

BENeplate, lang, lateraler 1,1 mm Draht (12 cm)

1:1

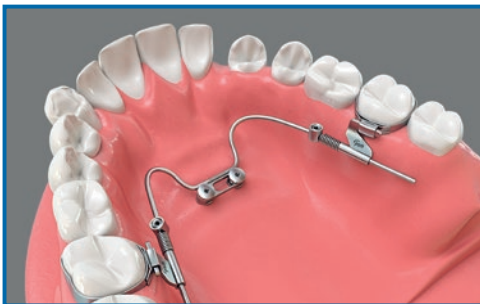
33-54471 1.1 mm stainless steel

BENeplate, short, incl. fixation screws, with 1.1 mm lateral wire (12 cm)

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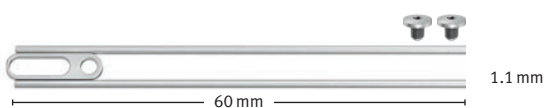
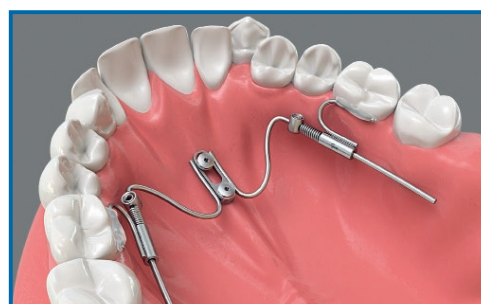
Selon le / Approvato da Dr. Thomas Banach,
Königsheim/Ts., Germany

■ Note:

All **BENeplates** are supplied with 2 fixation screws

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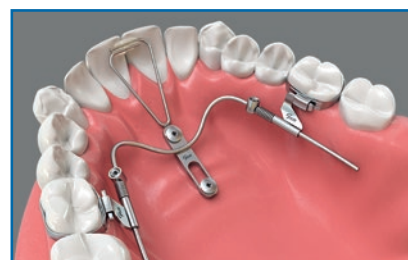
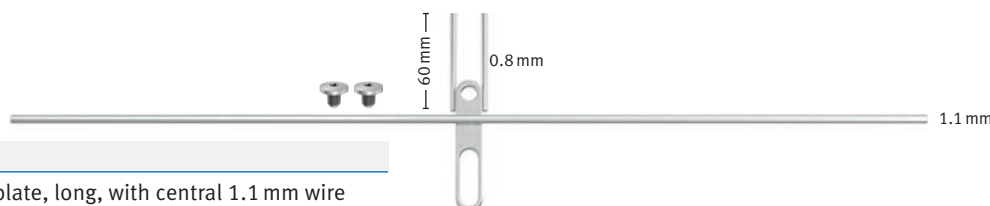
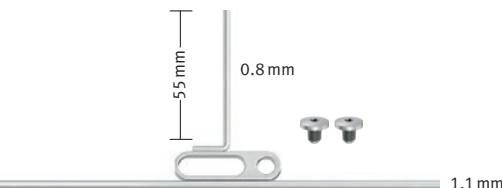
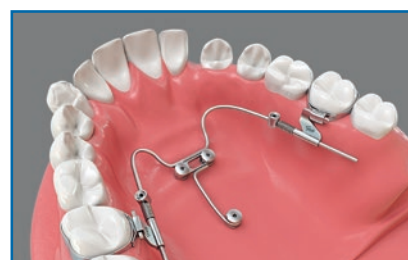
A device for molar distalization. The upper molars will be derotated, uprighted and distalized in only two or three months. By converting the system in a transpalatal bar, it can be used as an anchor for the molars. The anterior teeth can be retracted and aligned simply.

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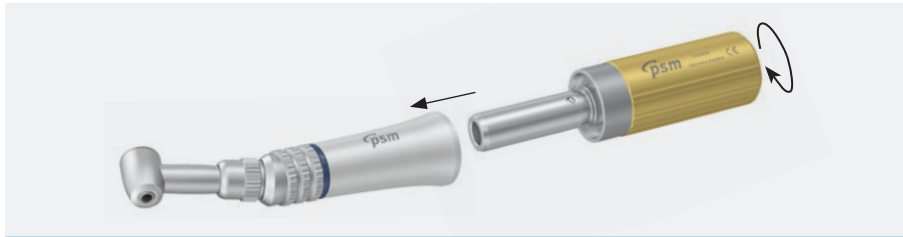
**GAMME DE TOURNEVIS
EN OPTION
OPTIONALE
SCHRAUBENDREHER**

Conditionnement: 1 pièce
VPE: 1 Stk.

The manually turned unit is connected to your existing contra-angled handpiece (blue 1:1) that enables its use like an angled screwdriver.

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10-63025

QB

Manually turned unit for contra-angled handpieces

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33-54704

QB

Manually turned unit mod. to Pauls, with adjustable torque from 0 – 40 Ncm

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33-18288

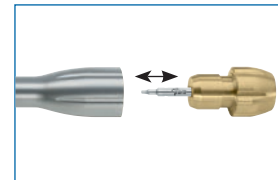
Laboratory Finger Screwdriver, fixed
Labor Finger Schraubendreher, fest

33-54520

Multi purpose handle for the use with instruments with dental shaft end.

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Note:

All products marked with **QB** can be used with **quattro** and **BENEFIT** screws!

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
BENEFIT Tournevis d'activation des écrous mobiles
BENEFIT Cacciavite attivatore

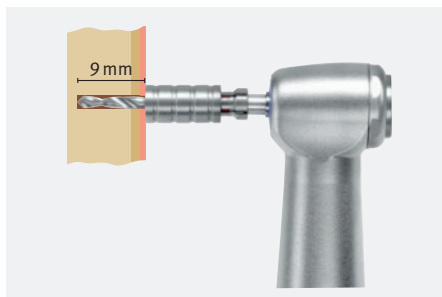Selon le / Approvato da Dr. Michael Schubert,
Regensburg, Germany


The drill stop prevents the clinician from unintentionally drilling too deep. They assure an accurate working length from 7 to 13 mm in 2 mm increments. By sliding the stop sleeve over the appropriate PSM drill, the working length is adjusted to the desired depth. Removal of the sleeve is easy using the buffer.

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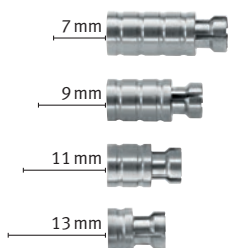33-18307

33-18300

Set, includes 4 sleeves (7 to 13 mm) without buffer

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10-61000

Screwdriver handle, only
Schraubendreher Handgriff, allein

*as long as stock lasts / solange der Vorrat reicht

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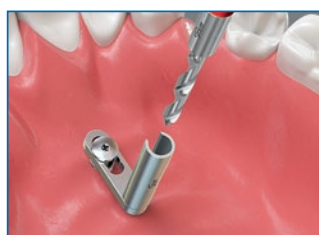
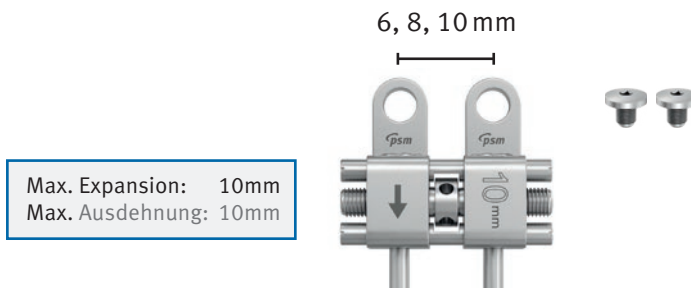
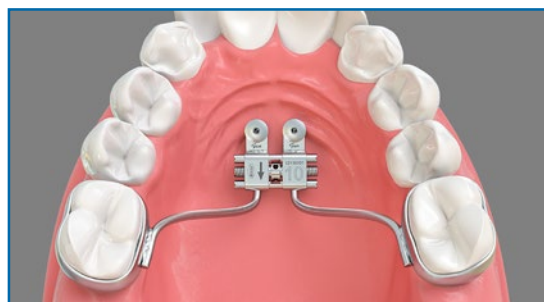
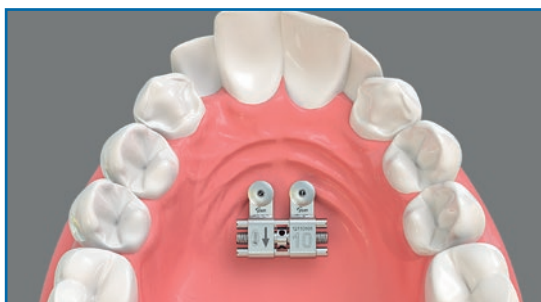
After using the **BENEFIT** Screw for the Hybrid Hyrax appliance very successfully in the last years the **bmx** now follows the Orthodontists request for an entirely **BENEFIT** mini-implant anchored appliance for Rapid maxillary expansion on young patients. Using the De Franco Screwguide (News 25) the distance and the parallelity of the **BENEFIT** Screws can be exactly determined, directly on the patient. An impression is no longer necessary. Alternatively visualisation can of course be made by means of digital imaging or traditional impression taking.

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Mesialsider NiTi Zugfedern, 2 Stk.

33-10760

Scanbody PEEK Standard
Scanbody PEEK Standard

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Developed in
cooperation with:
Entwickelt in
Zusammenarbeit mit:

Simon Graf,
Orthodontist
Belp, Switzerland

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Extension + 16mm

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33-18288

Laboratory finger screwdriver, fixed

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Recommendation
Empfehlung

■ Note:

*Set illustration can deviate from our set recommendation.

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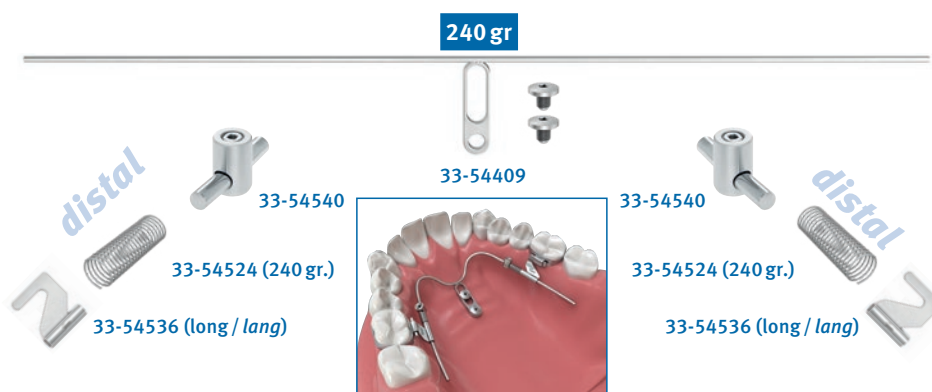
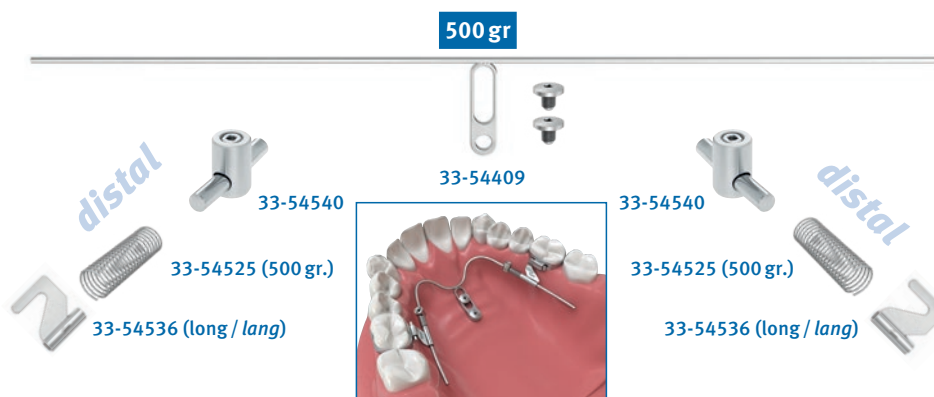
CATALOGUES
KATALOGE

benefit

For more information please refer to our **BENEFIT®** Manual or our website **www.psm.ms**

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quattro

The **quattro®** Mini Anchor System is known as one of the first and most successful orthodontic Mini-Implant Systems worldwide. Based on this system the **BENEFIT®** screws were developed. This enables the user to utilize the entire scope of orthodontic implantation with unmatched possibilities and compatible instrumentation.

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mentoplate

The Mentoplate-System provides a perfect solution for enhanced stability with special orthodontic bone anchored plates for class III correction. The System includes Titanium plates which are fixed to the bony surface with selfdrilling screws.

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BENEFIT®, quattro®, mentoplate®, BENESlider®, BENEplate®, bmx® are registered trademarks of PSM Medical Solutions, Germany.

BENEFIT®, quattro®, mentoplate®, BENESlider®, BENEplate®, bmx® sind eingetragene Marken von PSM Medical Solutions, Germany.

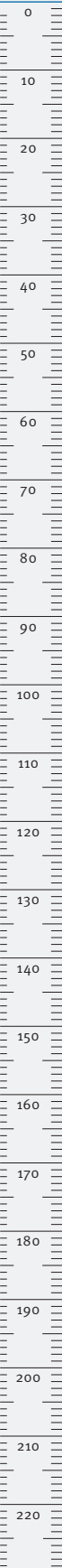
Ref: KAT-003

Ref: KAT-010

NOTES
NOTIZEN

Lined area for notes.

NOTES
NOTIZEN

The figure is a blank graph template. The vertical axis (y-axis) is on the left, with numerical labels from 0 to 220 in increments of 10. The horizontal axis (x-axis) is at the bottom, with 20 unlabeled grid lines. The graph area is a large rectangle with a light gray background. The axes are represented by thin black lines.

NOTES
NOTIZEN

Handwriting practice area with 25 horizontal lines.

Vertical ruler scale from 0 to 220 cm.

www.psm.ms – implants for professionals

BENefit[®].KFO

Distribution | Vertrieb:

Distribution in USA:

psm North America, Inc.

80900 Weiskopf
La Quinta, CA 92253
USA

Toll-free: 800-733-1622

service@psm-na.us
www.psm-na.us

Manufacturer | Hersteller:

psm MEDICAL SOLUTIONS GMBH

Gewerbestraße 10
78594 Gunningen, Germany

Phone +49 (74 24) 9 75 15-0
Fax +49 (74 24) 9 75 15-55

E-Mail info@psm.ms
Web www.psm.ms

© 2019 **psm** MEDICAL SOLUTIONS GMBH | Germany

Specifications and design are subject to change without prior notice. All pictures, photos and product descriptions are the property of **psm** MEDICAL SOLUTIONS GMBH. Utilization and copies by third parties have to be authorized by **psm** MEDICAL SOLUTIONS GMBH. All rights reserved!

All trademarks and brands mentioned in this brochure are subject to unrestricted regulations of the trade mark laws valid and right of ownership of the registered owner.

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