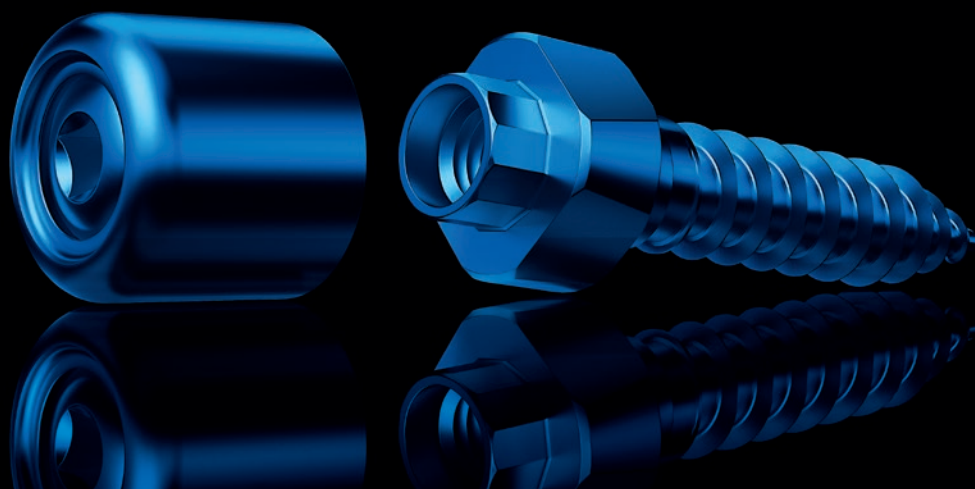
BENefit[®].KFO

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Skeletal anchorage in orthodontics

An important requirement for successful orthodontic treatment is stable anchorage. Conventional orthodontic anchorage can be limited by periodontal disease and/or missing teeth. Skeletal anchorage has been proven to be very effective. It can be an essential part of non-compliance therapy and broadens the options available to the orthodontic practitioner.

Mini-Implants are especially advantageous because of the minimally invasive insertion protocol and their relatively low cost.

However, limitations can occur when conventional Mini-Implant Systems are used for orthodontic anchorage. In collaboration with leading specialists, PSM has developed the unique **BENEFIT** TAD System for temporary fixation in the palate. **BENEFIT** offers high precision titanium implant screws and a large variety of prefabricated connecting parts such as **BENEplates**, abutments and palatal expanders.

With the introduction of the **BENEFIT** System PSM has considerably enhanced the therapeutic diversity of the treatments available for Mesialization, Distalization, RPE as well as uprighting or anchorage of the dentition.

With **BENEFIT**, PSM was the first company to offer a full range of skeletal anchorage appliances for the palate – a trend that has become state-of-the-art in Orthodontics.

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EINFÜHRUNG

■ Note:

All products marked with **QB** can be used with **quattro**® and **BENEFIT**® screws!

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BENEsider Video

The BENEsider

The most classical indication for skeletal anchorage using Mini-Implants with abutments is molar-distalization and/ mesialization in the maxilla. By means of the so called **"BENEsider"** the molars can be bodily distalized and/or mesialized using the Mesialslider. In many cases, tooth extraction or dental implants can be avoided.

To couple two **BENefit®** Mini-Implants, a **BENeplate** with welded wire is connected to the Implants and the **BENEsider** parts (mobilizer, springs, **BENetube**) are placed on the wire. Active force is applied by springs (240 g or 500 g) and activated with the Mobilizer.

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
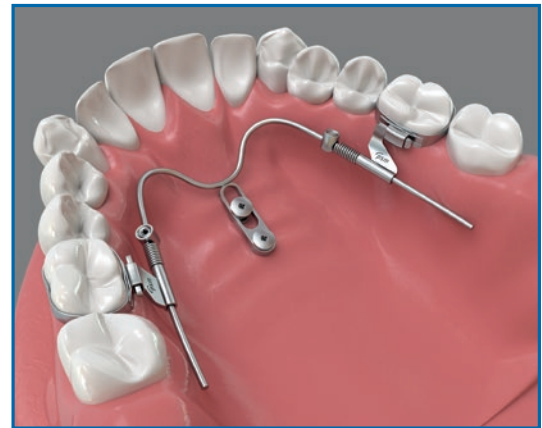
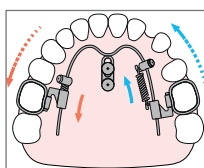
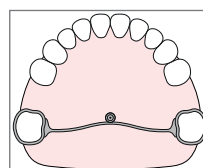
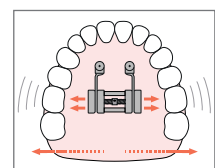
The Hybrid Hyrax®

The new Hyrax® Ring and Hyrax® Abutment (page 9) facilitate the construction and fixation of a Hybrid Hyrax® appliance. It allows welding or soldering with the Hyrax® Screw as well as little rotation movements when connecting the appliance to the BENEFit Screws. The Hyrax® Ring is supplied with 1 fixation screw each.

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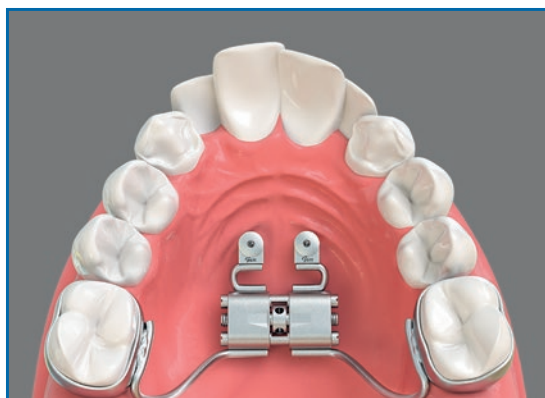Hybrid Hyrax Video

The bmx – BENEFit Maxillary Xpander

Additionally to the Hybrid Hyrax that is available for some years now the Orthodontists expressed their desire for an appliances that is exclusively anchored on mini-implants.

The new bmx System now offers the possibility for an enforced, non-surgical RPE with adolescent patients up to 13 years based on 2 paramedian anchored **BENEFit** miniscrews.

The pre-fabricated **bmX** expanders with 6 mm, 8 mm and 10 mm distance can be installed in the palate without an impression or additional laboratory work using the De Franco Parallel Guide (page 18).

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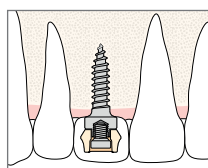
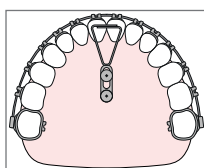
DENTAL

DENTAL

Screwdriver insert e.g. for abutment retaining screws, **BENEplate**
fixation screws and Mobilizer activation

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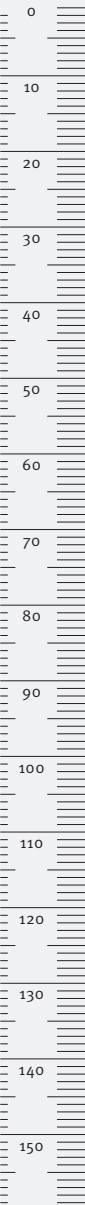
DENTAL

Extension, +16 mm
Verlängerung, + 16 mm

The extension of 16 mm works for all rotary instruments of the
quattro® and **BENEFIT®**-System with a latch type DENTAL attachment
(max. rpm 600).

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BENEFIT® Abutment with brackets

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33-54445

Abutment with loop acc. to Dr. S. Renger,
St. Louis, France

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Internal Thread for **BENEFIT** fixation screw
or the abutment screw.

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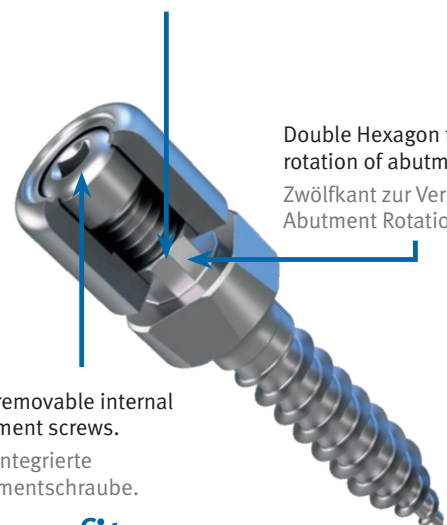
Double Hexagon to prevent
rotation of abutments.

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Non removable internal
abutment screws.

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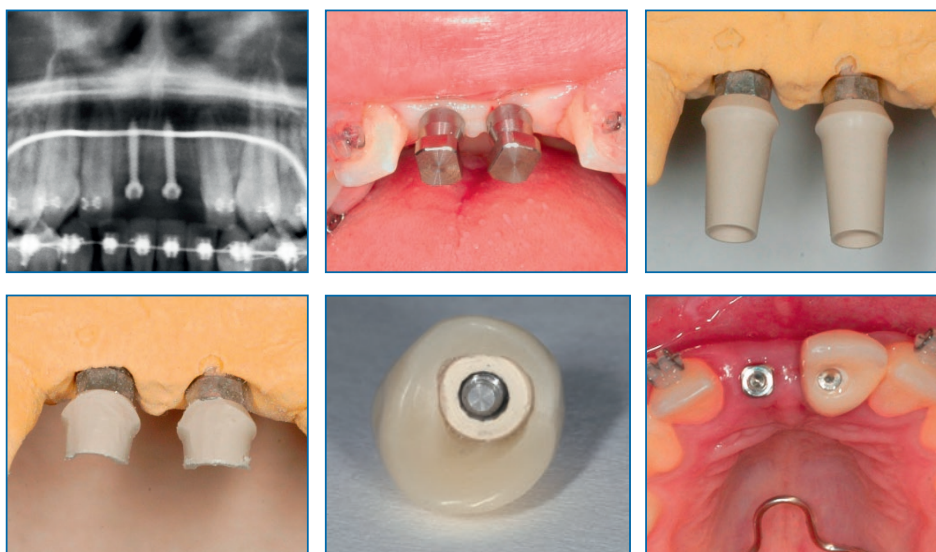
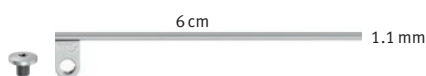
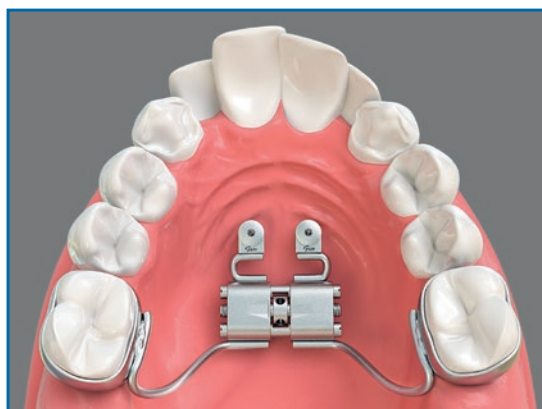
33-54463

BENEFIT® Hyrax® Abutment, 1 ea
BENEFIT® Hyrax® Abutment, 1 Stk.

The Hyrax® Ring and Hyrax® Abutment facilitate the construction and fixation of a Hybrid Hyrax® appliance. It allows welding or soldering with the Hyrax® Screw as well as little rotation movements when connecting the appliance to the BENEFIT Screws. The Hyrax® Ring is supplied with 1 fixation screw each.

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HYBRID HYRAX RPE HYBRID HYRAX GNE

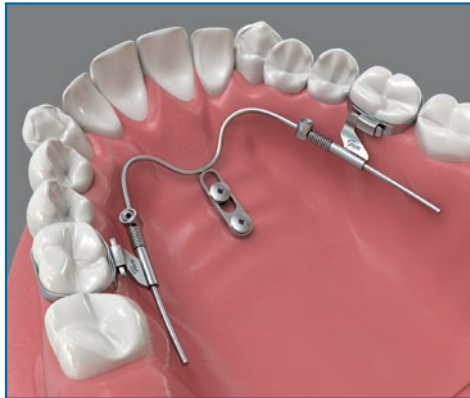
■ **Note:**
Further Hyrax Ring versions see on page 15.

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33-54545

Mesialtube acc. to Dr. Banach with wire 1.1 mm, 40 mm and hook

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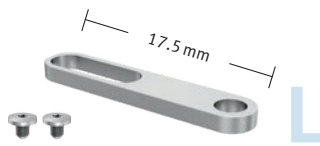
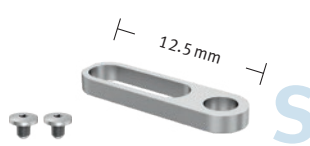
TMA

Note:

All **BENEplates** are supplied with 2 fixation screws

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Conditionnement: 1 pièce
Confezione: 1 pezzo

33-54472 1.1 mm stainless steel

BENEplate, long, with 1.1 mm lateral wire (12 cm)

BENEplate, lang, lateraler 1,1 mm Draht (12 cm)

paramedian

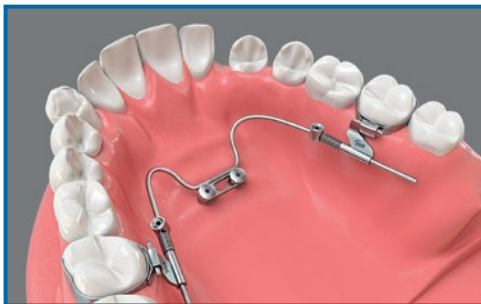
paramedian

33-54471 1.1 mm stainless steel

BENEplate, short, incl. fixation screws, with 1.1 mm lateral wire (12 cm)

BENEplate, kurz, inkl. Fixierschrauben, lateraler 1,1 mm Draht (12 cm)

BENEsider mesial/distal paramedian

BENEsider mesial/distal paramedian

33-54422 1.1 mm stainless steel

BENEplate, xs, incl. fixation screws, with 1.1 mm lateral wire (12 cm)

BENEplate, xs, inkl. Fixierschrauben, lateraler 1,1 mm Draht (12 cm)

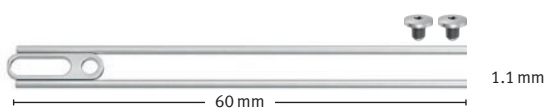
33-54475 0.8 + 1.1 mm stainless steel

Anka Plate, short, with 0.8 mm + 1.1 mm wire (12 cm)

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acc./nach Dr. George Anka, Tamashi, Tokyo, Japan


33-54477

BENEplate, short, with bi-lateral 1.1 mm wire 6 cm

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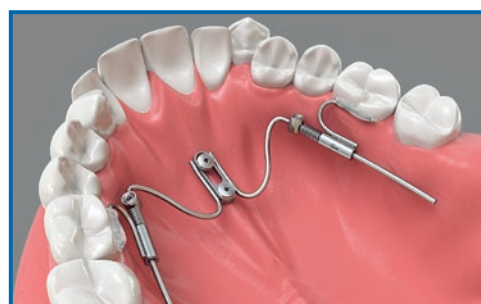
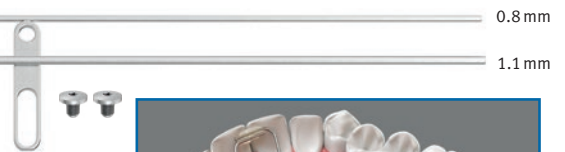
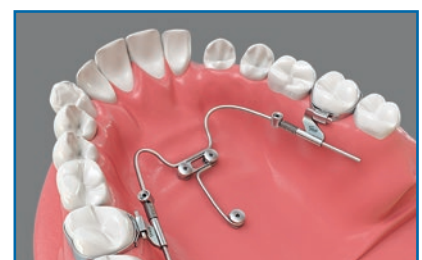
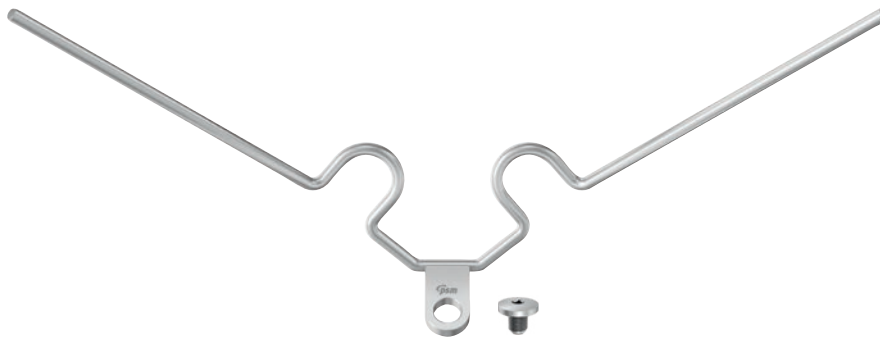
News 22

Developed in cooperation with:
Dr. Arnoud Schreurs, Orthodontist, Genk, Belgium.

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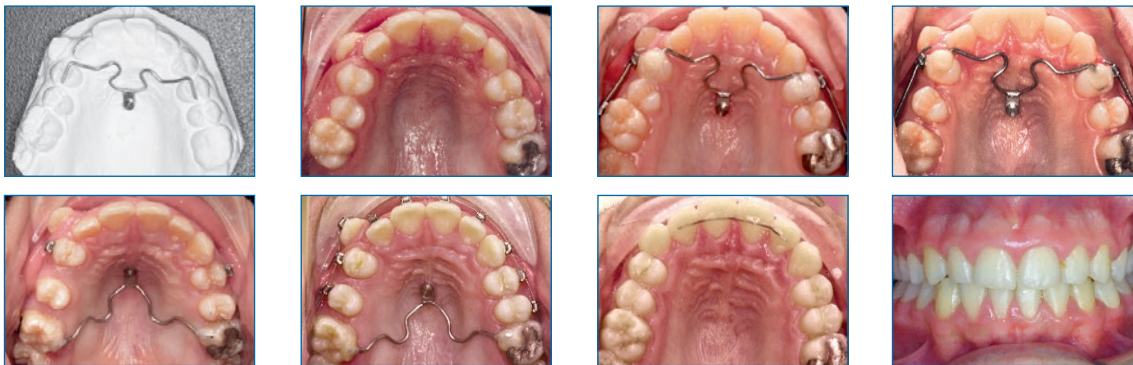
A device for molar distalization. The upper molars will be derotated, uprighted and distalized in only two or three months. By converting the system in a transpalatal bar, it can be used as an anchor for the molars. The anterior teeth can be retracted and aligned simply.

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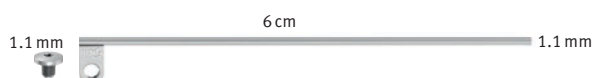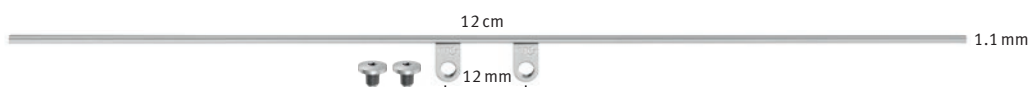
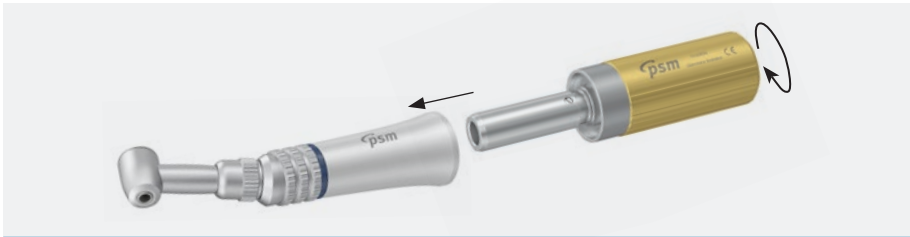
OPTIONAL SCREWDRIVERS
OPTIONALE
SCHRAUBENDREHER

Packaging: 1 ea.
VPE: 1 Stk.

The manually turned unit is connected to your existing contra-angled handpiece (blue 1:1) that enables its use like an angled screwdriver.

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10-63025 **QB**

Manually turned unit for contra-angled handpieces

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33-54704 **QB**

Manually turned unit mod. to Pauls, with adjustable torque from 0–40 Ncm

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33-18288

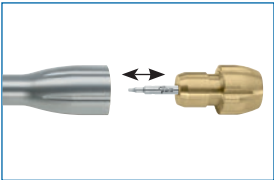
Laboratory Finger Screwdriver, fixed
Labor Finger Schraubendreher, fest

33-54520

Multi purpose handle for the use with instruments with dental shaft end.

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Note:

All products marked with **QB** can be used with **quattro** and **BENEFIT** screws!

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
BENEFIT Activating Screwdriver
BENEFIT Aktivierungsschraubendreher

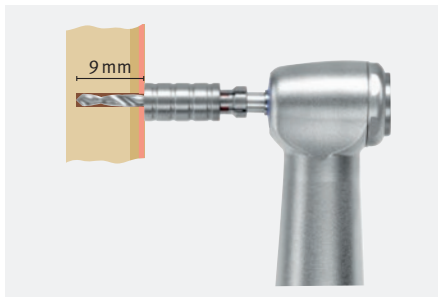acc./nach Dr. Michael Schubert, Regensburg, Germany


The drill stop prevents the clinician from unintentionally drilling too deep. They assure an accurate working length from 7 to 13 mm in 2 mm increments. By sliding the stop sleeve over the appropriate PSM drill, the working length is adjusted to the desired depth. Removal of the sleeve is easy using the buffer.

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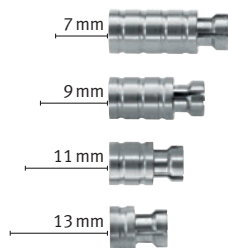33-18307

33-18300

Set, includes 4 sleeves (7 to 13 mm) without buffer

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10-61000

Screwdriver handle, only
Schraubendreher Handgriff, allein

*as long as stock lasts / solange der Vorrat reicht

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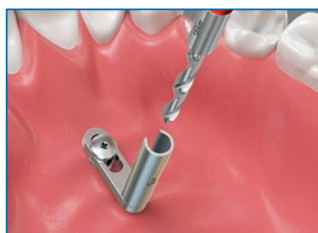
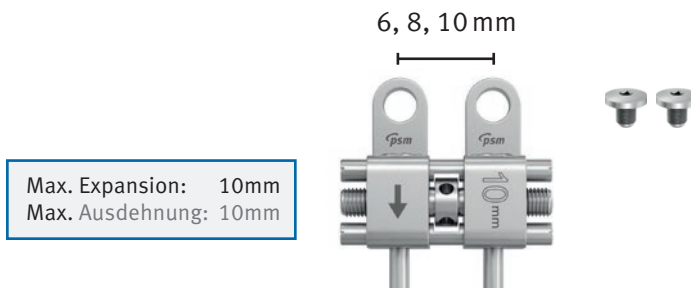
33-54576


bmx *Benefit Maxillary Xpander*

BENEFit
MAXILLARY XPANDER
BENEFit
MAXILLARY XPANDER

33-57506	6 mm
33-57508	8 mm
33-57510	10 mm

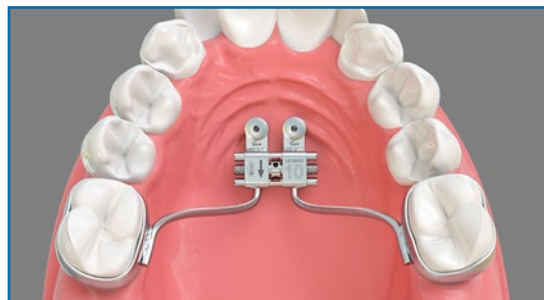bmx, palatal expander incl. 2 fixation screws
bmx, Gaumennahtspreizer inkl. 2 Fixierschrauben

After using the **BENEFit** Screw for the Hybrid Hyrax appliance very successfully in the last years the **bmx** now follows the Orthodontists request for an entirely **BENEFit** mini-implant anchored appliance for Rapid maxillary expansion on young patients. Using the De Franco Screwguide (News 25) the distance and the parallelity of the **BENEFit** Screws can be exactly determined, directly on the patient. An impression is no longer necessary. Alternatively visualisation can of course be made by means of digital imaging or traditional impression taking.

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Mesialsider NiTi Zugfedern, 2 Stk.

33-10760

Scanbody PEEK Standard
Scanbody PEEK Standard

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SCANBODIES SCANBODIES

Material: PEEK
Packaging: 1 ea.
VPE: 1 Stk.

BENEFIT® screws will be i.e. available for virtual case planning within the following software libraries:

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- 3 shape
- Onyxceph TAD match
- EASY DRIVER
- En cours

As a further step into digitalization of the BENEFIT®-System these scanbodies were developed in cooperation with orthodontist Simon Graf (Belp, Switzerland) to ensure visualization of a detailed rendering.

The Scanbody Standard is used right after implant insertion. The Scanbody mini can be screwed onto the implants even after several months of treatment

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Developed in cooperation with:
Entwickelt in Zusammenarbeit mit:

Simon Graf,
Orthodontist
Belp, Switzerland

MOBILHOLDER
MOBILHOLDER

for Beneslider, Mesialslider

Abb. / fig. 3

The new Mobilholder is intended to move the **BENEFIT®** Mobilizers into the final position prior to tightening the set screw for Distalization (fig. 1 / 2) or Mesialization (fig. 3).

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Mobilizer with Hook

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33-54543

Micro Mobilizer for wires from 0.5 to 1.2 mm

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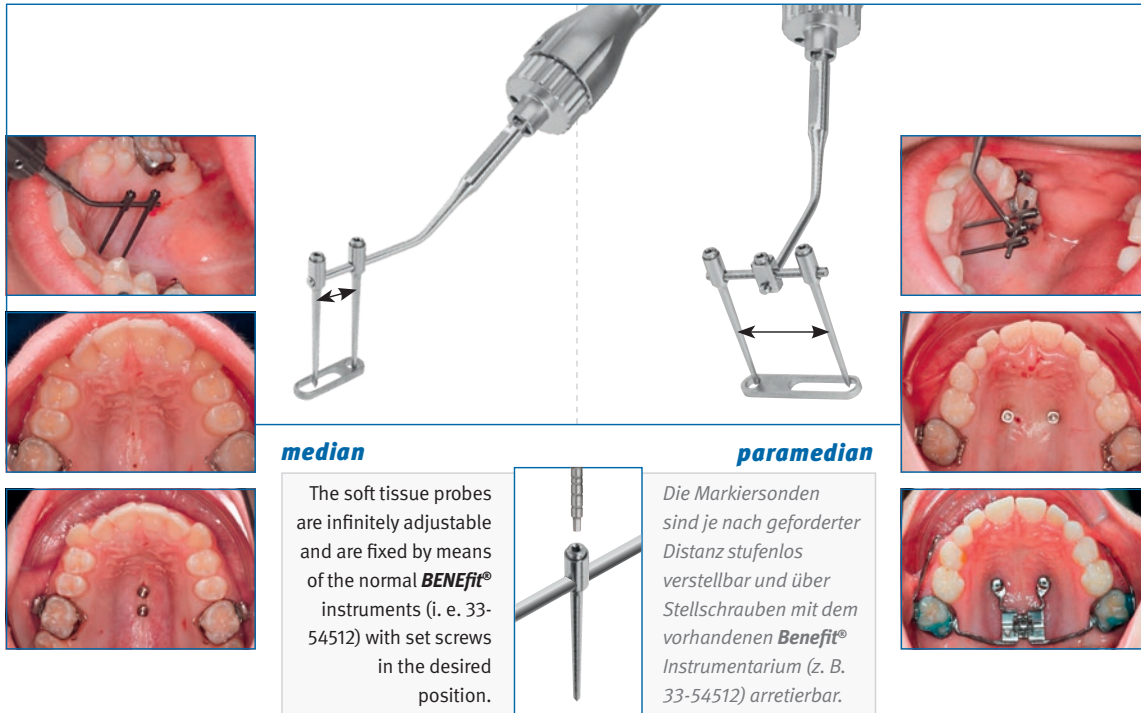
SCREW LOCATION PROBE ZIELSONDE

median

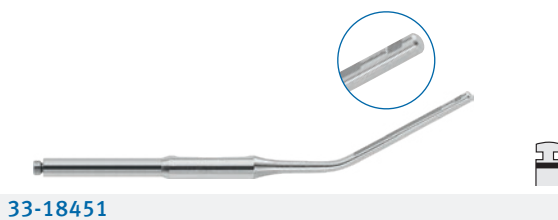
The soft tissue probes are infinitely adjustable and are fixed by means of the normal **BENEFIT®** instruments (i. e. 33-54512) with set screws in the desired position.

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The screw location probe acc. to Dr. Schlichter identifies the target points for the median or paramedian placement of the **BENEFIT®** Screws. The precise positioning of the **BENEFIT®** Implants improves the location and parallelism with each other. It also increases stability and simplifies the chairside bending of the plate. Additionally the lab time is reduced due to less bending.

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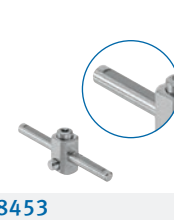
Developed in cooperation with
Dr. Kai Schlichter, Dres. Wer-
ling & colleagues, Offenbach a.
d. Q., Germany.

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BENESLIDER TITANIUM
BENESLIDER TITANIUM

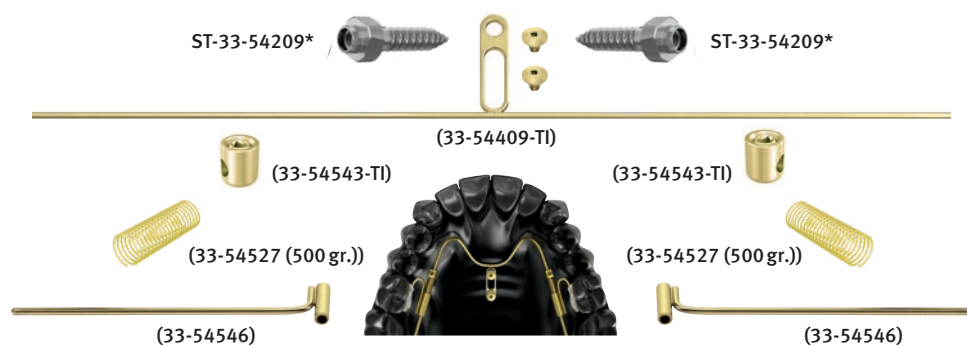
Material: Titanium
Material: Titan
Packaging: Set only
VPE: nur als Set

The parts of the new **PSM BENESlider Titanium** are manufactured from surgical titanium according to American and European Standards providing biocompatible material for nickel or steel allergic patients. The specially gold anodized titanium surface identifies the titanium parts and offers an increased titaniumoxid layer for perfect biocompatibility.

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BENESlider Titanium Set complete incl. 2x Benefit® Screws 2.0 x 9 mm*

BENESlider Titan Komplettsset inkl. 2x Benefit® Schrauben 2.0 x 9 mm*

*For different screws sizes please notice this in your order.

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**The BENESlider titanium set is only available as a complete set. Parts are not sold separately.

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Cordless Battery Screwdriver with Torque Calibration System for the fixation of **quattro** and **BENEFIT** Miniscrews.

The iSD 900 will guarantee precise torque control due to the unique – easy to use – torque calibration system (10 – 40 Ncm). Time-saving and precise, riskfree insertion and tightening of screws and abutments.

The complete Set contains a reliable and user friendly handpiece for Dental attachments. Reverse mode is indicated with sound. Standard rechargeable AAA Ni-Mh batteries can be used.

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iSD900

BATTERY SCREWDRIVER
AKKU SCHRAUBENDREHER

FEATURES VORTEILE

- 10 – 40 Ncm torque in 1 or 5 increments
- 5, 10, 15, 25 rpm turning speed
- Reverse mode with sound
- Calibration facility
- Standard rechargeable batteries

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■ **Note:**

All products marked with **QB** can be used with **quattro®** and **BENEFIT®** screws!

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L

33-55002

BENEFIT® Starter Instrument Set
without implants
consisting of:

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Extension + 16mm

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33-18288

Laboratory finger screwdriver, fixed

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Recommendation
Empfehlung

■ Note:

*Set illustration can deviate from our set recommendation.

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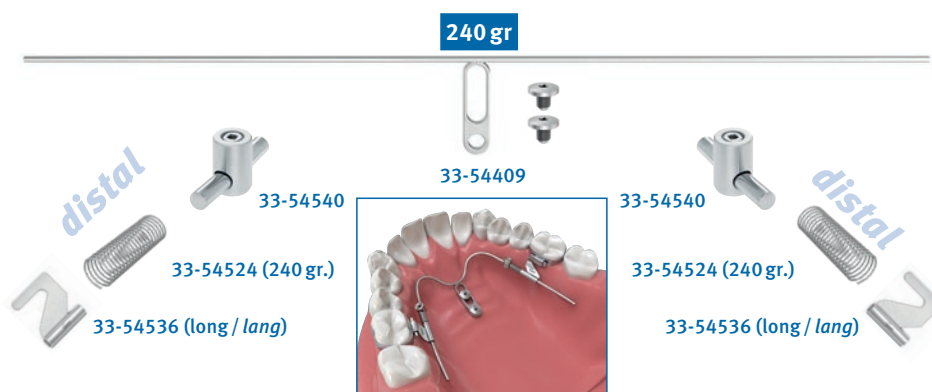
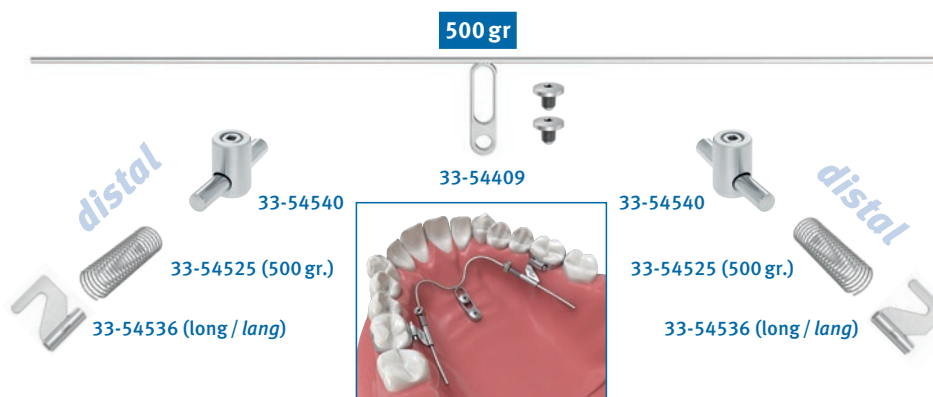
Due to the large possibility of Set variations available with the BENEFIT System we only have shown the most common variations in this catalogue. Your local PSM distribution partner will be pleased to give you further assistance.

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CATALOGS KATALOGE

Even more innovative PSM products are presented in our other catalogues. Please request it from your local PSM distribution partner or download from our website www.psm.ms.

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benefit

For more information please refer to our **BENEFIT®** Manual or our website www.psm.ms

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quattro

The **quattro®** Mini Anchor System is known as one of the first and most successful orthodontic Mini-Implant Systems worldwide. Based on this system the **BENEFIT®** screws were developed. This enables the user to utilize the entire scope of orthodontic implantation with unmatched possibilities and compatible instrumentation.

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Ref: KAT-003

mentoplate

The Mentoplate-System provides a perfect solution for enhanced stability with special orthodontic bone anchored plates for class III correction. The System includes Titanium plates which are fixed to the bony surface with selfdrilling screws.

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Ref: KAT-010

BENEFIT®, quattro®, mentoplate®, BENESlider®, BENEplate®, bmx® are registered trademarks of PSM Medical Solutions, Germany.

BENEFIT®, quattro®, mentoplate®, BENESlider®, BENEplate®, bmx® sind eingetragene Marken von PSM Medical Solutions, Germany.

NOTE

[illegible]

0	
10	
20	
30	
40	
50	
60	
70	
80	
90	
100	
110	
120	
130	
140	
150	
160	
170	
180	
190	
200	
210	
220	

NOTES

NOTE

0
10
20
30
40
50
60
70
80
90
100
110
120
130
140
150
160
170
180
190
200
210
220

[illegible]

NOTES

NOTE

Handwriting practice area with 25 horizontal lines.

Vertical ruler scale from 0 to 220 cm.

www.psm.ms – implants for professionals

BENefit[®].KFO

Distribution | Vertrieb:

Distribution in USA:

psm North America, Inc.

80900 Weiskopf
La Quinta, CA 92253
USA

Toll-free: 800-733-1622

service@psm-na.us
www.psm-na.us

Manufacturer | Hersteller:

psm MEDICAL SOLUTIONS GMBH

Gewerbestraße 10
78594 Gunningen, Germany

Phone +49 (74 24) 9 75 15-0
Fax +49 (74 24) 9 75 15-55

E-Mail info@psm.ms
Web www.psm.ms

© 2019 **psm** MEDICAL SOLUTIONS GMBH | Germany

Specifications and design are subject to change without prior notice. All pictures, photos and product descriptions are the property of **psm** MEDICAL SOLUTIONS GMBH. Utilization and copies by third parties have to be authorized by **psm** MEDICAL SOLUTIONS GMBH. All rights reserved!

All trademarks and brands mentioned in this brochure are subject to unrestricted regulations of the trade mark laws valid and right of ownership of the registered owner.

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